INSTRUCTION MANUAL

Why using mamivac® nipple shields?

Many mothers use conventional round shaped nipple shields when having sensitive and sore nipples. These should be used for a short time only, because they have influence on the natural way of breastfeeding. The round shape covers mum's skin, and it may happen that the baby refuses to suck when mum is trying to breastfeed again without a nipple shield. This is because the baby does not identify its mother due to the irritation by the nipple shield. The special shape of **mamivac®** nipple shields avoid this by reason of not interrupting the contact between mother and baby. Therefore the baby reminds the natural smell of its mother.

Use:

Clean your breast before using **mamivac®** nipple shields. A short massage has the effect that nipples stand upright and make the use of **mamivac®** nipple shields much easier. Turn the soft edges up and press the **mamivac®** nipple shield as close as possible to your nipple. Press the soft edges afterwards to your skin. Wetting the **mamivac®** nipple shield supports the attachment to your skin. Make sure to place the **mamivac®** nipple shield with the flat side up that your baby's nose has direct contact to your skin while breastfeeding. Then the baby identifies your smell and feels safe. Do not use Vaseline or other cremes. For further information regarding the use please ask your lactation consultant or midwife.

Cleaning and sterilisation:

mamivac® nipple shields are made of thin, transparent and tasteless silicone. Boil **mamivac®** breast shields before each usage at least 3 to 5 minutes. Retain the instruction manual! This product is not a toy. Keep it out of reach of children.

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KaWeCo GmbH