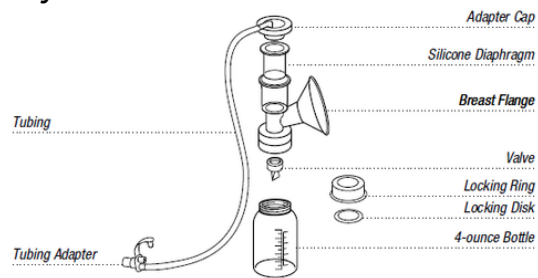
Trusted by Hospitals, Loved by Moms.

Ameda HygieniKit H Series Milk Collection System

Intended Use - Single patient

The HygieniKit Milk Collection System is intended to facilitate the expression and collection of breast milk from a lactating woman for the purpose of feeding the collected milk to a child.

Set Up and Assembly

Cleaning

Unless told otherwise by a healthcare provider, follow these cleaning instructions: Always wash hands well with soap and water before handling the Ameda HygieniKit and other Ameda products. Do not use solvents or abrasives.

Before First Use

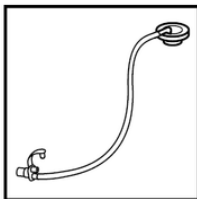
If your Ameda HygieniKit is marked “STERILE,” open and use.


If your Ameda HygieniKit is NOT marked “STERILE,” before first use:

- Boil all parts—except tubing, adapter cap, and tubing adapter—for 20 minutes.
- Remove all parts from water right after boiling.
- Allow to air dry on a clean, dry surface before assembly and use.


After Every Use

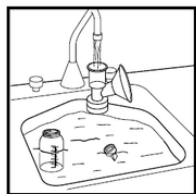- Take apart the Ameda HygieniKit to clean.
- Rinse any milk residue from pump parts with cool water.
- Wash all parts—except tubing, adapter cap, and tubing adapter—in warm, soapy water.

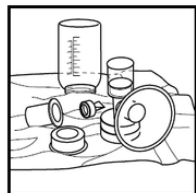- Rinse well with warm, clean water.
- Allow parts to air dry on clean towel or drying rack before assembly and next use.
- Cover parts with clean towel if not planning to use again right away.

Breast flanges and bottles can be washed on the upper rack of a standard household dishwasher away from heating elements. Wash diaphragms and valves gently by hand.

Do not insert anything into valve while cleaning.

STERILE **EO** **MD**
**Includes/
Contient/Incluye
25mm**

**MADE WITHOUT
BPA**

EN **Universal Single HygieniKit® Milk Collection System 25mm**
STERILE - Contents sterile, unless package is opened or damaged.
Do not use if package is damaged.

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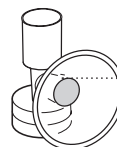
Please see Instructions for Use to ensure proper and safe use of the Ameda HygieniKit

- (1) Instructions/Instructions/Instructions/Gebrauchsanleitung/Istruzioni/Instruções/Gebruiksaanwijzingen/Vejledning/Käyttöohjeet/**Οδηγίες/Talimatlar/Instruçiuni**
- (1) Storage Bag/Sac de rangement/Bolsa de almacenamiento/Aufbewahrungsbetel/Custodia/Bolsa de arrumação/Bewaarzakje./Opbevaringspose/ Säilytyspussi/**Σακούλα Αποθήκευσης/Saklama Poşeti/Pungă de stocare**
- (1) Adapter Cap/Embout de l'adaptateur/ Tapa del adaptador/Adapterdeckel/ Tappo dell'adattatore/Tampa do Adaptador/ Verbindingskapje/Adapterhætte/Sovittimen korkki/**Πώμα προσαρμογέα/Adaptör Kapağı/Сарац adaptor**
- (1) Silicone Diaphragm/Diaphragme de silicone/Diafragma de silicona/Silikonmembrane/Diaframma in silicone/Diafragma de Silicone/Siliconendiafragma/Silikonediafragma/Silikonikalvo/**Διάφραγμα σιλικόνης/Silikon Diyafram/Diafragmă de silicon**
- (1) Breast Flange (30.5mm)/Manchon pour seins (30,5mm)/Embudo de lactancia (30,5)/ Brustglocke (30,5 mm)/Campana per il seno (30,5 mm)/Flange Mamária/Kolfstuk (30,5 mm)/Brystflange (30,5 mm)/Rintalaippa (30,5mm)/ **Χοάνη (30,5mm)/ Göğüs Flaşı (30,5 mm)/Cupă colectoare (30,5 mm)**
- (2) Valves/Valves/Válvulas/Ventile/valvole/Válvulas/ventiel/ventile/venttiilit/ **βαλβίδες/Valfler/supape**
- (1) Bottle 120mL/Bouteille de 120mL/Botella de 120mL/Flasche (120 mL)/Flacone (120 mL)/Frasco (120mL)/Fles (120mL)/Flaske (120 ml)/Pullo (120 mL)/ **Φιάλη (120mL)/ Şişe (120 mL)/Sticlă (120 mL)**
- (1) Tubing Adapter/Adaptateur pour tubes/Adaptador del tubo/Schlauchadapter/Adattatore del tubo/Adaptador do Tubo/ Verbindingsstuk voor slang/Slangeadapter/Letkusovitin/ **Προσαρμογέας σωλήνα/Hortum Adaptörü/Adaptor tuburi**
- (1) Tube/Tubulure/Tubo/Schlauch/Tubo/ Tubo/Slang/Slange/Putki/**Σωλήνας / Tüp/ Tub/**
- (1) Locking Ring/Locking Disc/Anneau de verrouillage/Disque de verrouillage/Anillo de cierre/Disco de cierre/Verriegelungsring/Verriegelungsscheibe/Anello di bloccaggio/Disco di bloccaggio/Aro de Bloqueo/Disco de Bloqueo/Afsluitring/Afsluitschijf/Låsering/Låseskive/Lukitusrengas/ Lukituslevy/ **Δακτύλιος ασφάλισης /Δίσκος ασφάλισης/ Kilitleme Halkası/Kilitleme Diski Inel de blocare/Disc de blocare**

30.5mm

Contents sterile, unless package is opened or damaged. Do not use if package is damaged.

Distributed in Canada by:
Distribué au Canada par :
Mothers Choice Products Ltd
2133-11871 Horseshoe Way
Richmond, BC V7A 5H5
1.800.604.6225
www.motherschoiceproducts.com

Distributed in Australia by:
Midmed Pty Ltd.
Unit 4, 62 Borthwick Avenue
Murarrie, QLD 4172
61 (7) 3348 9155

Distribution in other countries: For product information or feedback, call your local distributor or location where you purchased the product. For a listing of distributors in your country, please visit www.ameda.com.

Transport/Storage:
Transport/Rangement:
Transporte/almacenamiento:

Manufactured for:
Ameda, Inc.
485 Half Day Road, Ste. 302
Buffalo Grove, IL 60089
USA

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Assembled in Mexico with US and foreign components. | Assembled in Mexico with US and foreign components. | Assembled in Mexico with US and foreign components. | Assembled in Mexico with US and foreign components. | Assembled in Mexico with US and foreign components.

STERILE EO MD

EN **Universal Dual HygieniKit® Milk Collection System 25mm**
STERILE - Contents sterile, unless package is opened or damaged.
Do not use if package is damaged.

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The Ameda Logo, Ameda and HygieniKit are trademarks and/or registered trademarks of Ameda, Inc. in the United States and other countries. Le logo d'Ameda Logo, Ameda et HygieniKit sont des marques de commerce ou des marques déposées d'Ameda, Inc. aux États-Unis et dans d'autres pays.