Trusted by Hospitals, Loved by Moms.

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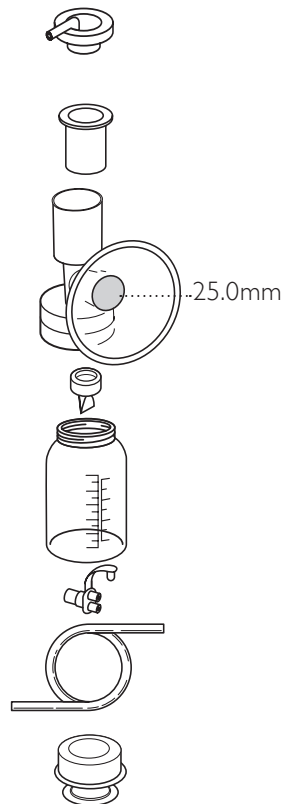
Please see Instructions for
Use to ensure proper and safe
use of the Ameda HygieniKit

- (1) Instructions/Instructions/Instructions/Gebrauchsanleitung/Istruzioni/Instruções/
Gebruiksaanwijzingen/Vejledning/Käyttöohjeet/Οδηγίες/Talimatlar/Instrucțiuni
- (1) Storage Bag/Sac de rangement/Bolsa de almacenamiento/Aufbewahrungsbetel/
Custodia/Bolsa de arrumação/Bewaarzakje./Opbevaringspose/ Säilytyspussi/
Σακούλα Αποθήκευσης/Saklama Poşeti/Pungă de stocare
- (1) Adapter Cap/Embout de l'adaptateur/ Tapa del adaptador/Adapterdeckel/ Tappo
dell'adattatore/Tampa do Adaptador/ Verbindingskapje/Adapterhætte/Sovittimen
korkki/Πώμα προσαρμογέα/Adaptör Kapaşı/Capac adaptor
- (1) Silicone Diaphragm/Diaphragme de silicone/Diafragma de silicona/Silikonmembrane/
Diaframma in silicone/Diafragma de Silicone/Siliconendiafragma/Silikonediafragma/
Silikonikalvo/Διάφραγμα σιλικόνης/Silikon Diyafram/Diafragmă de silicon
- (1) Breast Flange (25.0mm)/Manchon pour seins (25,0mm)/Embudo de lactancia
(25,0)/ Brustglocke (25,0 mm)/Campana per il seno (25,0 mm)/Flange Mamária/
Kolfstuk (25,0 mm)/Brystflange (25,0 mm)/Rintalaippa (25,0mm)/ Χοάνη (25,0 mm)/
Göğüs Flanşı (25,0 mm)/Cupă colectoare (25,0 mm)
- (2) Valves/Valves/Válvulas/Ventile/valvole/Válvulas/ventiel/ventile/venttiilit/ βαλβίδες/
Valfler/supape
- (1) Bottle 120mL/Bouteille de 120mL/Botella de 120mL/Flasche (120 mL)/Flacone
(120 mL)/Frasco (120mL)/Fles (120mL)/Flaske (120 ml)/Pullo (120 mL)/ Φιάλη (120mL)/
Şişe (120 mL)/Sticlă (120 mL)
- (1) Tubing Adapter/Adaptateur pour tubes/Adaptador del tubo/Schlauchadapter/
Adattatore del tubo/Adaptador do Tubo/ Verbindingsstuk voor slang/Slangeadapter/
Letkusovitin/ Προσαρμογέας σωλήνα/Hortum Adaptörü/Adaptor tuburi
- (1) Tube/Tubulure/Tubo/Schlauch/Tubo/ Tubo/Slang/Slange/Putki/Σωλήνας / Tüp/ Tub/
- (1) Locking Ring/Locking Disc/Anneau de verrouillage/Disque de verrouillage/Anillo de
cierre/Disco de cierre/Verriegelungsring/Verriegelungsscheibe/Anello di bloccaggio/
Disco di bloccaggio/Aro de Bloquei/Disco de Bloqueo/Afsluitring/Afsluitschijf/
Låsering/Låseskive/Lukitusrengas/ Lukituslevy/ Δακτύλιος ασφάλισης /Δίσκος
ασφάλισης/ Kilitleme Halkası/Kilitleme Diski Inel de blocare/Disc de blocare

Contents sterile, unless package is opened or damaged. Do not use if package is damaged.

Distributed in Canada by:
Distribué au Canada par :
Mothers Choice Products Ltd
2133-11871 Horseshoe Way
Richmond, BC V7A 5H5
1.800.604.6225
www.motherschoiceproducts.com

Distributed in Australia by:
Midmed Pty Ltd.
Unit 4, 62 Borthwick Avenue
Murarrie, QLD 4172
61 (7) 3348 9155

Distribution in other countries: For product information or feedback, call your local distributor or location where you purchased the product. For a listing of distributors in your country, please visit www.ameda.com.

Transport/Storage:
Transport/Rangement:
Transporte/almacenamiento:

Manufactured for:
Ameda, Inc.
485 Half Day Road, Ste. 302
Buffalo Grove, IL 60089
USA

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STERILE EO MD

EN Universal Single HygieniKit® Milk Collection System 28.5mm
STERILE - Contents sterile, unless package is opened or damaged.
 Do not use if package is damaged.

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Die diaphragm in this product creates a barrier protecting liquids collected in the collection bottle system from penetration of bacteria,^{1,2} and viruses (Hepatitis B and C and Human Immunodeficiency Virus HIV)^{1,2,3} from the pump and tubing in the event of contamination of these components during a prior use or by a prior user.

Note that the Ameda HygieniKit neither removes bacteria and viruses (Hepatitis B and C and Human Immunodeficiency Virus HIV) from the breast milk being collected into the collection bottle nor does the Ameda HygieniKit make it safe for infected mothers to feed their infants with milk collected using this device.

^{1,2} Testing performed in situ using actual HygieniKits and SMB Piston Breast Pump with suspension of phiX174 Bacteriophage and Escherichia coli, Staphylococcus pneumoniae and Pseudomonas aeruginosa. Procedure was adapted from ASTM 1671-97b "Standard Test method Resistance of materials Used in Protective Clothing to Penetration by Blood-borne Pathogens Using Phi-X174 Bacteriophage as a test System"

² The use of this product neither guarantees or warranties against the transmission of Hepatitis Band C and Human Immunodeficiency Virus HIV.

IMPORTANT NOTE: ONLY USE THE AMEDA HYGIENIKIT MILK COLLECTION SYSTEMS ON AMEDA ELECTRIC BREAST PUMPS.

NOTE: Before using the Ameda HygieniKit Milk Collection System, check the parts for cracks, chips, tears, color changes, or wear. They can affect the function. If parts need replacing due to defects, please call your local distributor or the location where you purchased the pump. For a listing of distributors in your country, please visit www.ameda.com.

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The Ameda Logo, Ameda and HygieniKit are trademarks and/or registered trademarks of Ameda, Inc. in the United States and other countries.