



INSTRUCTIONS FOR USE IN ENGLISH - Important instructions below - please read carefully and retain.

Autoclavable Reusable bottles - Instructions for use - Product Codes: 14093 & 14093/W

For Hospitals and Milk Banks please confirm with your Infection Control Department that you can re-use the bottles. It will depend on how you anticipate washing & sterilizing the bottles & teats.

The graduations are a guide only & meet the requirements of BS EN14350:2020. They should not be used as a measuring device.

Directions before use - Do not use if there are signs of damage or weakness to either the primary packaging or the product. Inspect all components before use, discard if any imperfections are visible. Inspect the feeding teat before each use and pull the feeding teat in all directions. Throw away at the first signs of damage or weakness. Remove all packaging before using the bottle/teat. Before each subsequent use clean carefully to ensure hygiene.

Usage limitations - This product is unsuitable for use with liquid above 80°C. This product is suitable for pasteurisation (PA product codes must not be submerged) (Excludes teats). This product is suitable for freezing to -25°C. Do not fill above the maximum graduation marked on the bottle. Bottles with foil-sealed caps are only suitable for 5 sealing cycles – a new cap must be used each time (do not attempt to re-attach a used foil seal).

Sterilisation information and limitations - Wash and sterilise all component immediately after use. For Hospitals and Milk banks this product is only suitable for 5 uses/sterilisation cycles. For Home use continue to reuse until the product has visible signs of wear & tear and dispose of when this is visible. This product is suitable for autoclaving (Max 121°C for 20min), dishwasher, microwave sterilisation, cold water sterilisation & steam sterilisation. Note: The caps on product codes 14093 & 14093/W are not suitable for Autoclaving. If using a dishwasher, choose a mild detergent (below pH 9.0) to avoid damage or premature failure. After sterilisation, carefully check each component for damage or distortion before re-use, discarding if any imperfections are visible.



Single use bottles – Instructions for use - Product codes:11200, 14000/14100, 14003/14103, 14001/14101, 14002/14102,

The graduations are a guide only & meet the requirements of BS EN14350:2020. They should not be used as a measuring device.

Directions before use – For products 11200 do not use if there are signs of damage or weakness to either the primary packaging or the product. Inspect each component before use, discard if any imperfections are visible. Remove all packaging before using the product.

Usage limitations - These products are unsuitable for use with liquid above 80°C. These products are suitable for pasteurization (Not 11200). Do not fill above the maximum graduation marked on the bottle. Bottles are a single use product and shall be disposed of after first use. Products are suitable for freezing down to -25°C.

Note: Medicare Colgate guarantees these products when used according to the above instructions. Medicare Colgate accepts no liability for loss or damage and will not replace or reimburse for damages when they are caused by using this product in any other manner.