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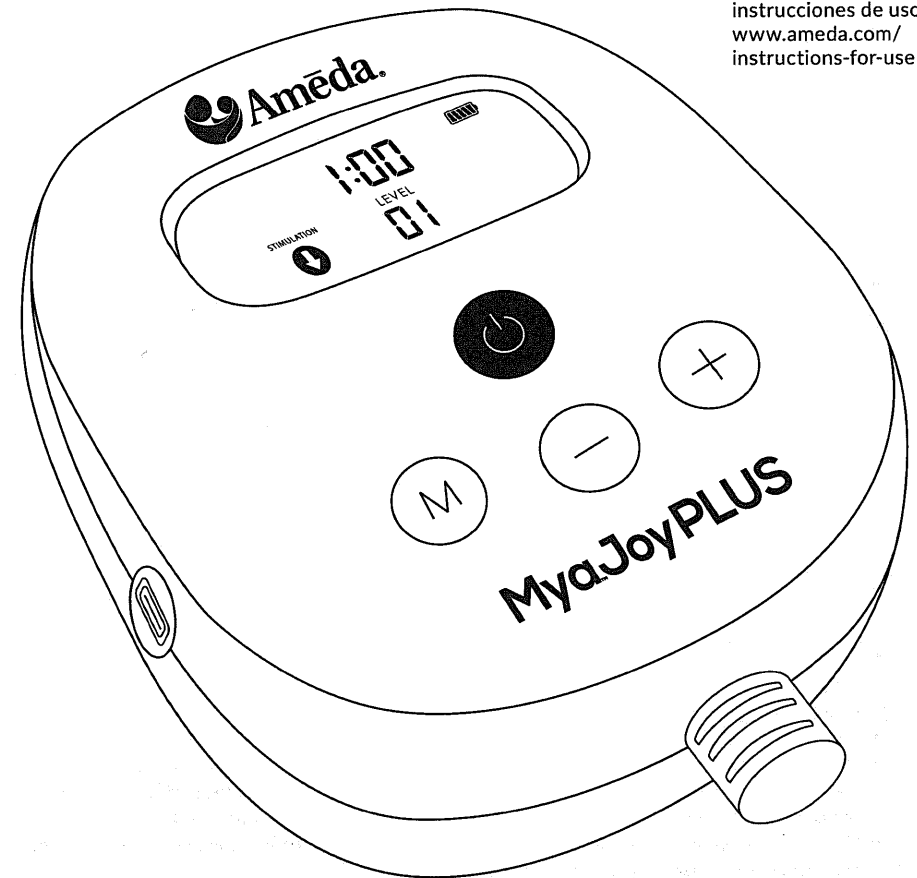
Follow QR Code to
view Instructions
for Use video
[www.ameda.com/
instructions-for-use](http://www.ameda.com/instructions-for-use)

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WELCOME TO THE AMEDA FAMILY!

We are excited to help you meet your breastfeeding goals along your breastfeeding journey with your baby. If you have any questions regarding your Ameda breast pump, our certified ParentCare team is available to help you. Have your breast pump and parts nearby when you call. Ameda ParentCare is available by phone during business hours on weekdays at 866.992.6332.

For more information about breastfeeding, breast pumping and other Ameda products visit www.ameda.com. For more product information or feedback in other countries, visit www.ameda.com for a list of distributors outside of the U.S.

IMPORTANT SAFETY GUIDELINES

Carefully read this Instructions for Use (Quick Start) prior to using your new Ameda breast pump and follow these safety guidelines. Refer to the online instruction manual for complete instructions and product overview.

Download the full instruction manual at www.ameda.com/myajoy

CAUTION & WARNINGS

- DO NOT expose device to rain, moisture, dripping, or splashing.
- NEVER operate this product if it has a damaged cord or plug; if it is not working properly; if it has been dropped or damaged, or becomes wet.
- Keep cord away from heated surface.
- ALWAYS inspect power adapter wires prior to use for damage or exposed wire. If damage is found, immediately discontinue use of power adapter and call Ameda ParentCare at 866.992.6332 or your local distributor.
- Before each use visually inspect the individual components. If damage is observed, please discontinue use until the parts have been replaced.
- This product is intended for use by a single user only and should not be shared between users.
- Clean and sanitize all parts that come in contact with your breast and breast milk prior to first use.
- DO NOT wash or boil tubing,
- DO NOT wrap cord around power adapter body.
- Make sure the voltage of the power adapter is compatible with the power source.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast flange with your finger and remove the breast flange from your breast.
- This product contains a lithium-ion battery. The battery or internal components cannot be replaced. DO NOT attempt to open the pump base.
- If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
- If irritation or discomfort occurs, discontinue use and see a doctor.
- DO NOT attempt to remove the breast flange from your breast while pumping. If you are not feeling comfortable or feeling pain with pumping, turn off the pump, break the seal between the breast and breast flange with your finger, and remove breast flange from your breast gently.
- Use only microwave bags specifically labeled for disinfecting breast pump components and refer to the microwave bag labeling for further instructions.
- STRANGULATION HAZARD: Keep tubing and power adapter cords out of reach of children or persons with disabilities.
- CHOKING HAZARD: Small parts, NOT for children under 3 years.

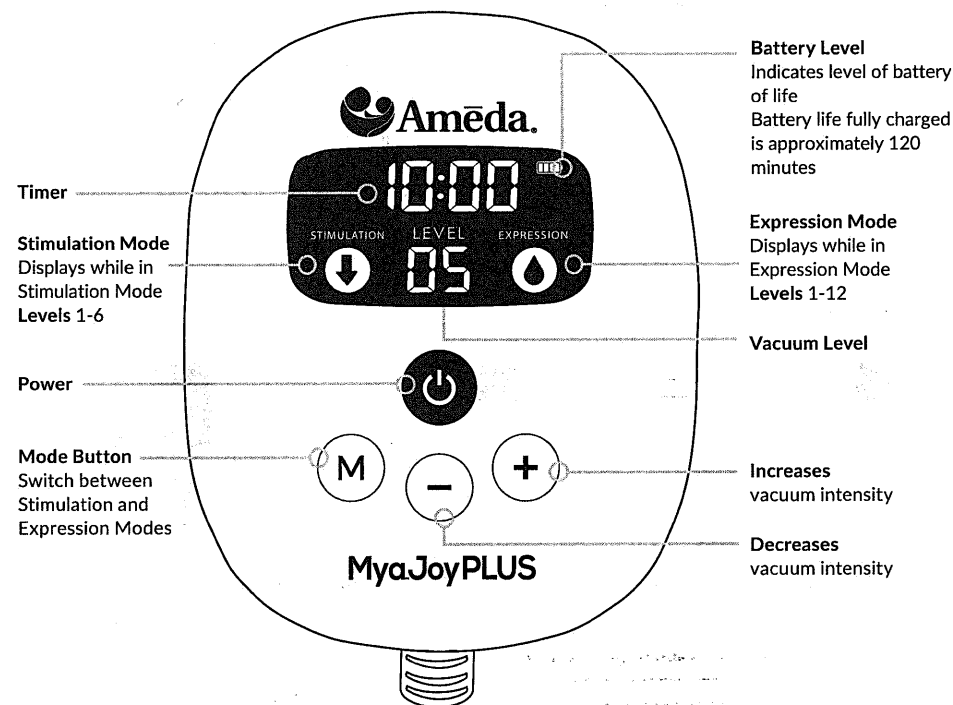
GETTING STARTED

Refer to the online instruction manual for complete instructions and product overview.



Download the full instruction manual at www.ameda.com/myajoy

1. Before first use, sanitize all parts (refer to page 4 for parts description) – **except adapter cap, tubing, tubing adapter and pump motor** – by boiling water for 20 minutes or by using the new CleanEase™ Steam Bag. Remove all parts from water after boiling or microwaving. Dry completely before assembly.
2. Understand the display icons and button controls (page 3).
3. Connect unit to wall power using AC Adapter to charge battery. Battery icon stops blinking when fully charged.
4. Assemble the pump kit and connect to pump (page 4).
5. Follow the step-by-step instructions on how to breast pump (page 5).


DISPLAY ICONS & BUTTON CONTROLS

KIT ASSEMBLY FOR ELECTRIC PUMPING

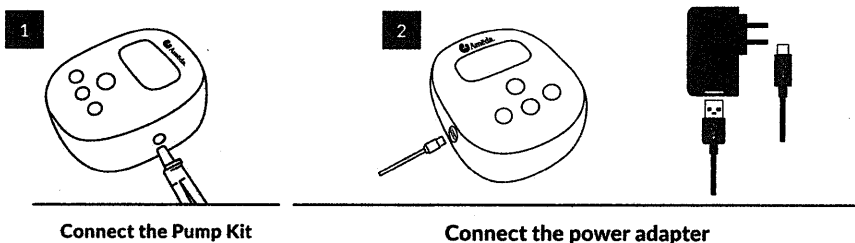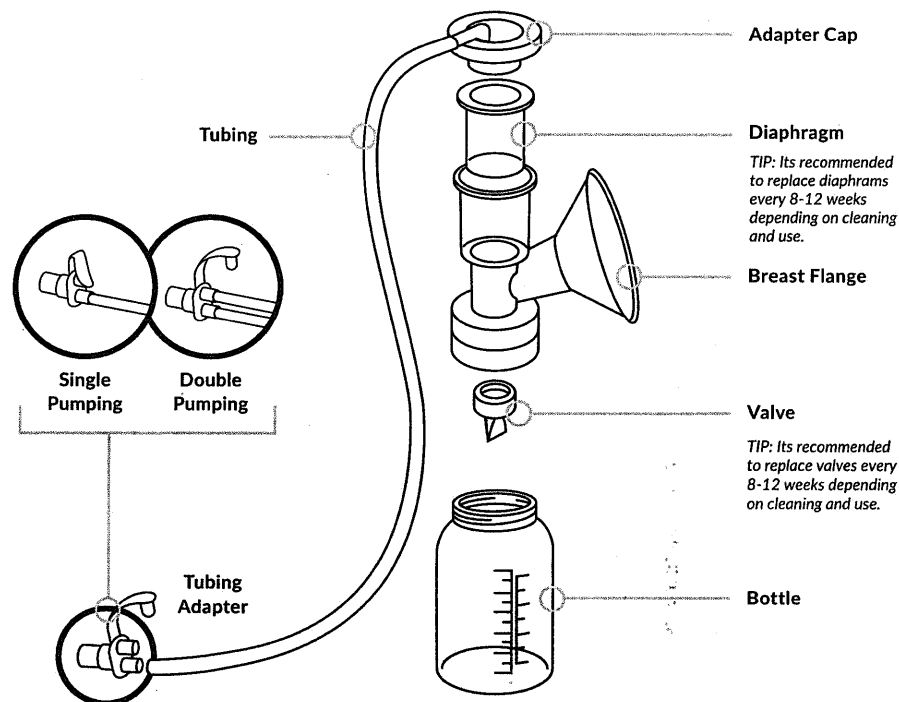TO BEGIN PUMPING


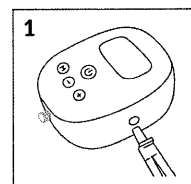
STEP 1: INSERT TUBING

Insert Tubing adapter into the right side of the pump base, then turn the pump slightly to the right, and gently push it to make sure it connects firmly.

A. For **DOUBLE PUMPING**: Both tubes should be attached to the tubing adapter.

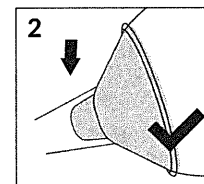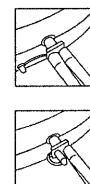B. For **SINGLE PUMPING**: Remove one tube from the tubing adapter and close one side using the attached plug.


STEP 2: PLACE THE FLANGE ON BREAST

Center flange on breast to create seal.


Check breast flange fit, if pumping causes discomfort, a larger or smaller breast flange size may be needed. See section on how to properly fit your flange.

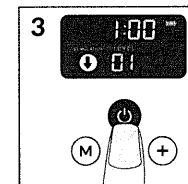See section below on how to properly fit your flange.

STEP 3: POWER ON THE PUMP

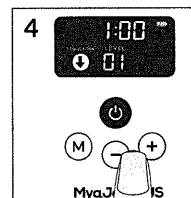Press the Power button to turn on the device. It will begin in Stimulation Mode.


STEP 4: ADJUST VACUUM LEVEL

Adjust the Stimulation Vacuum Level between 1 and 6 to your preferred comfort level. Never set to uncomfortable level.


TIP: Start off with the suction low, and then gradually increase the suction setting to your most comfortable level. You may find that the level of suction that you are comfortable with varies from day to day, during the day and even during the same expression session.

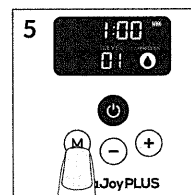NOTE: After two minutes, the pump will automatically switch to Expression Mode unless you manually adjust prior.

STEP 5: CHANGE MODE

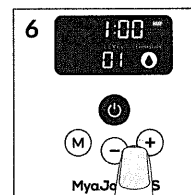
Once your milk starts to flow, press the Mode button to switch into Expression Mode.

STEP 6: ADJUST VACUUM LEVEL

Adjust the Expression Vacuum Level between 1 and 12 to your preferred comfort level. Never set to uncomfortable level.

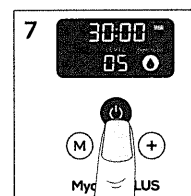TIP: Start off with the suction low, and then gradually increase the suction setting to your comfort level. You may find that the level of suction that you are comfortable with varies from day to day, during the day and even during the same expression session.


STEP 7: END PUMPING SESSION

When you wish to stop expressing milk, first press the Power button, then remove milk collection kit from your breast.

TIP: After pump is off, if necessary, INSERT a finger between breast and breast flange to break suction.


How to Fit Your Flange

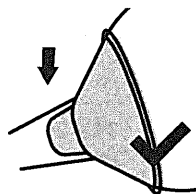Check breast flange fit, if pumping causes discomfort a larger or smaller breast flange size may be needed.


GOOD FIT

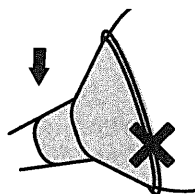During pumping, your nipple moves freely in the breast flange tunnel. You see space around the nipple. Not much areola is drawn into the tunnel with the nipple.

FLANGE TOO SMALL

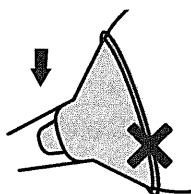During pumping, some or all of your nipple rubs against the sides of the breast flange tunnel.

FLANGE TOO LARGE

During pumping, more areola is drawn into the breast flange with your nipple. Your areola may rub against the side of the breast flange tunnel.

Breast Milk Storage Guidelines

NOTE: When thawing frozen milk, label as thawed when completely thawed (i.e. no ice crystals present). Use the time when completely thawed to base acceptable time limits for use rather than when it is taken from the freezer. These guidelines for storage and thawing of breast milk are a recommendation. Contact your lactation consultant or breastfeeding specialist for further information.

	Room Temperature 77°F (25°C)	Time in Refrigerator 39°F (4°C)	Time in Freezer 0°F (-18°C)
Fresh Expressed or Pumped	Up to 4 hours	Up to 4 days	6 Months - Best 12 Months - Acceptable
Thawed, Previously Frozen	1-2 Hours	Up to 1 day (24 hours)	Do Not Refreeze
Leftover from Feeding (baby did not finish bottle)	Use within 2 hours after the baby is finished feeding		

These guidelines are for healthy full-term babies and may vary for premature or sick babies. Check with your health care provider. Find more breastfeeding resources at: WICBreastfeeding.fns.usda.gov and www.cdc.gov/breastfeeding/

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Distributed in Canada by:
Distribue' au Canada par:
Distribuido en Canadá por:

Mothers Choice Products Ltd.
2133-11871 Horseshoe Way
Richmond, BC V7A5H5
800.604.6225

Distribution in other countries:
For product information or feedback, call
your local distributor or location where
you purchased the product. For a listing of
distributors in your country, please visit
www.ameda.com

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The Ameda Logo, Ameda, Ameda Mya and/or registered trademarks of Ameda, Inc. in the United States and other countries.

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To register your Ameda Mya Joy Plus pump:
Please visit [ameda.com/register](https://www.ameda.com/register) and input the serial number (SN) from the bottom of your pump.

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Le logo Ameda, Ameda, Ameda Mya et/ou marques déposées d'Ameda, Inc. aux États-Unis et dans d'autres pays.