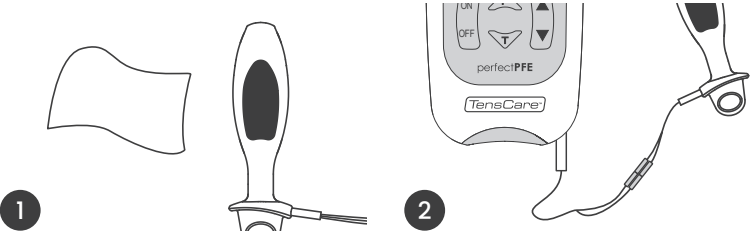
perfect**PFE**


Instructions for Use
Please read carefully

TensCare™


Quickstart Guide

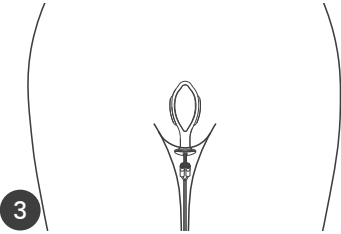1 Clean the probe with a wipe or with warm soapy water.

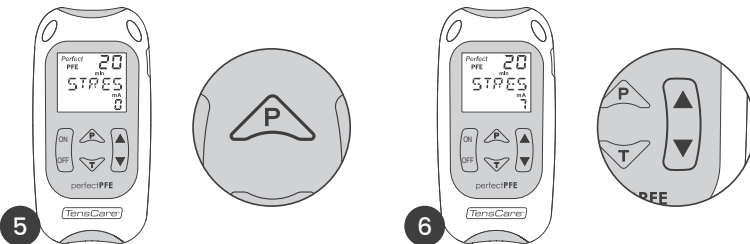3 Insert the probe into the vagina with the metal plates facing outwards.

5 Select a programme by pressing the P button.

6 Start the session by increasing the intensity.

Contents

1. Device Description	3
1.1. Introduction	3
1.2. Intended purpose	3
1.3. Kit contents	3
1.4. Treatment programmes	3
1.5. Controls and display	5
2. How to Use	6
2.1. Device setup	6
2.2. Preparing for a session	6
2.3. Inserting the probe	7
2.4. Starting a session	7
2.5. During a session	8
2.6. After a treatment	10
2.7. Treatment guidance summary	11
2.8. Alternative electrode treatment	12
2.9. Alternative anal probe treatment	12
2.10. Kegel exercises	13
3. Troubleshooting	14
4. Device Care	15
4.1. Cleaning	15
4.2. Storage	16
4.3. Battery replacement	16
4.4. Probe and lead wire care	16
4.5. Replacement parts/accessories	17
5. Technical Specification	18

1. Device Description

1.1. Introduction

The Perfect PFE is a clinically proven electrical muscle stimulator designed to treat urinary incontinence and improve your pelvic floor health. It uses gentle electrical signals delivered through a small vaginal probe to stimulate and contract the pelvic floor muscles. This promotes muscle strength and pelvic floor muscle and bladder control, helping to reduce or eliminate urine leakage. Pelvic floor stimulation may also help to enhance your sexual health.

Perfect PFE can also be used to improve chronic pelvic pain

When used as intended, the Perfect PFE can provide the following clinical benefits:

- Reduced frequency of stress incontinence episodes
- Reduced frequency of urge incontinence episodes
- Improved pelvic floor muscle strength and bladder control
- Relief from chronic pelvic pain

1.2. Intended purpose

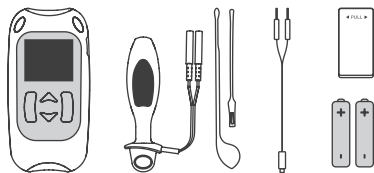
The Perfect PFE is a medical device designed to be used in the home healthcare environment to treat symptoms of stress, urge and mixed urinary incontinence. Perfect PFE is also designed to help improve chronic pelvic pain. Perfect PFE is suitable

for use by adults (18 years+) who can control the device and understand the instructions.

Do not use the device for any other purpose.

1.3. Kit contents

- 1 x Perfect PFE device
- 1 x Liberty Loop vaginal probe (X-VPL)
- 1 x Lead wire (L-CPT)
- 2 x AA batteries
- 1 x Storage pouch
- 1 x Instructions for Use

1.4. Treatment programmes

Perfect PFE can be used to treat incontinence and chronic pelvic pain.

Incontinence

Perfect PFE has different Electrical Muscle Stimulation (EMS) treatment programmes depending on the type of incontinence you have.

Stress Incontinence

Stress incontinence is when urine leaks during activities like coughing, sneezing, laughing, or exercising. This happens because the muscles supporting the bladder and urethra weaken, often due to childbirth, aging, or surgery. As a result, the

muscles can't control urine release, leading to accidental leakage.

Urge Incontinence

Urge incontinence is when there is a sudden, intense urge to urinate, followed by involuntary urine leakage. It occurs when the bladder muscles contract too often or uncontrollably, even when the bladder isn't full. This can be caused by infections, neurological conditions, or other underlying health issues.

Mixed Incontinence

Mixed incontinence is when a person has both stress incontinence and urge incontinence, meaning they leak urine during activities like coughing or exercising and also have sudden, strong urges to urinate that they can't control.

Pelvic pain

Perfect PFE's Pain programme uses Transcutaneous Electrical Nerve Stimulation (TENS) to alleviate chronic pelvic pain. TENS is a clinically proven and safe treatment option for providing pain relief.

Perfect PFE uses low-frequency TENS. This promotes the release of endorphins, the body's natural pain killer. Endorphin Release builds up over 40 minutes and will last several hours after stimulation has finished.

Select from the following treatment programmes:


Prog.	Description	Default treatment time	Usage guidance
Stress	The stimulation triggers muscle contractions, which helps to strengthen them.	20 minutes	Use the device once daily, with noticeable improvements typically occurring within four weeks.
Urge	The gentle stimulation soothes the bladder muscle, reducing involuntary contractions and preventing unexpected bladder emptying.	Continuous	Use the device once daily, with noticeable improvements occurring in as little as two weeks.
Mixed	Combination of stress and urge programmes to treat mixed incontinence.	20 minutes	Use the device once daily, with noticeable improvements typically occurring within four weeks.

Tone	After-care programme for maintaining pelvic floor health once improvements have been made. This programme is also recommended for sexual health.	20 minutes	Use twice per week.
------	--	------------	---------------------

Pain	TENS stimulation provides relief from chronic pelvic pain.	Continuous	Use when experiencing pain. While you can use it for extended periods, maximum relief is typically achieved after about 40 minutes.
------	--	------------	---

i **Note:** Perfect PFE incontinence treatment is most effective when complemented by Kegel Exercises. These involve voluntary contractions and relaxing of the pelvic floor muscles. See section 2.9 for guidance on performing Kegel Exercises.

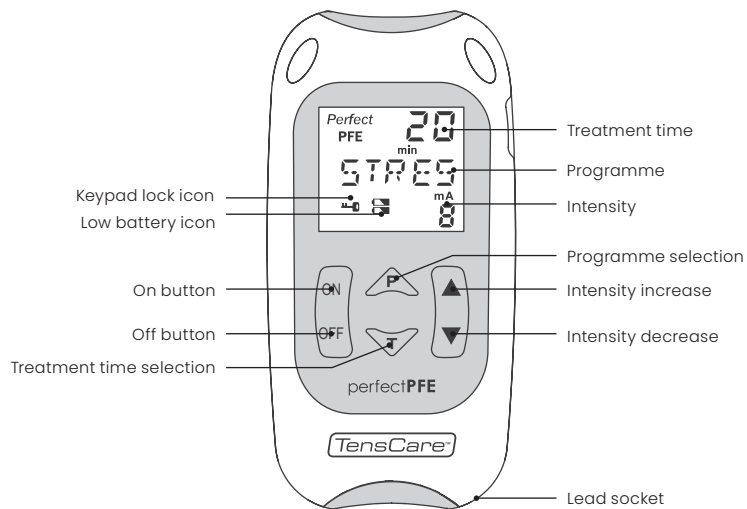1.5. Controls and display


2. How to Use


2.1. Device setup

Install the batteries

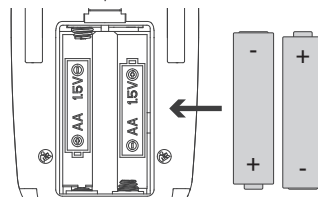1. Remove the battery cover on the rear of the device.
2. Insert the batteries, ensuring that the positive (+) and negative (-) ends match the corresponding terminals in the compartment.

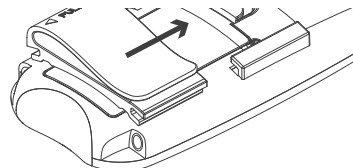
3. Replace the battery cover.
4. To use the belt clip, slide the belt clip into the rear channels of the device.

Connect the leads

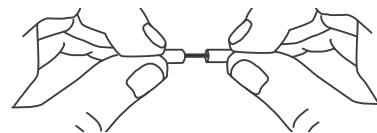
1. Insert the plastic end of the lead wire into the socket on the rear of the device.
2. Push the metal pins of the lead wires into the sockets on the probe lead.

Connecting the pointer

The Liberty Loop probe features a detachable pointer, which serves as a visual indicator for monitoring the effectiveness of your pelvic floor muscle contractions.

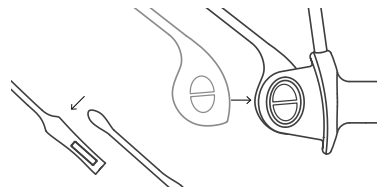1. Assemble the two halves of the pointer.
2. Insert the base of the pointer into the circular hole at the end of the probe, ensuring the pointer is positioned at the following angle:

2.2. Preparing for a session

1. Before inserting the probe, it is advised to empty your bladder.
2. Clean the probe before using for the first time and after each use. Clean with either an alcohol-free antibacterial wipe such as TensCare Wipes or by washing with warm soapy water.
3. Lubricate the metal plates of the probe with water or a water-based lubricant (such as TensCare's Go Gel).

- Get into a comfortable position, such as sitting down or lying down with your knees bent and your feet flat.

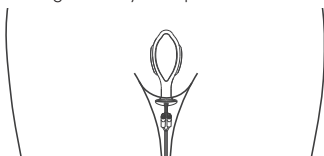
2.3. Inserting the probe

- With the probe connected to the device and adequately lubricated, gently insert it into the vagina.


Warning: Ensure the Perfect PFE is switched off before inserting the probe.

- Ensure the metal sides of the probe are facing toward your hips.

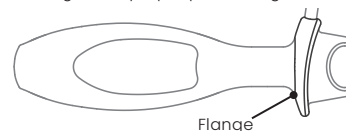- Avoid inserting the probe too deeply; the flange at the end should remain outside the vagina for proper positioning.

Note: Wearing tight-fitting underwear or jeans can help keep the probe in place and maintain proper contact during the session.

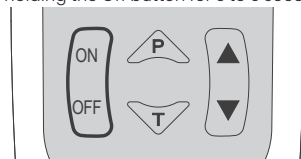
2.4. Starting a session

Turning on / off

- Turn the device on by pressing and

holding the On button for 3 to 5 seconds.

- Turn the device off by pressing and holding the Off button for 3 to 5 seconds.

Note: The device will automatically turn off if the intensity setting is left at zero for more than 5 minutes.

Programme selection

- Select the desired programme by pressing the P button. (See section 1.4 Treatment programmes).

Each time you change the programme the intensity level will revert to zero.

When turning on, the device will remember the last programme used.

Treatment time selection

- To select the treatment time, press the T button. The min icon will flash on the display.

- Press ▲▼ to change the treatment time from either 10, 20, 30, 45, 60, 90 minutes or continuous.

- Press T again to confirm the treatment time.

It is recommended to use the default treatment times for each programme.

Note: As with regular muscle exercise, the longer the training session the greater the muscle fatigue and time for recovery.

Note: Treatment time cannot be adjusted in the Mixed programme. This is set at 20 minutes.

Note: For Urge programme, the default treatment time is set as continuous. Start with a daily treatment of 20 minutes and increase the duration if you do not notice improvements after 2 weeks.

Note: The pain programme can be used for as long as necessary. While you can use it for extended periods, maximum relief is typically achieved after about 40 minutes. Longer use may cause muscle ache.

Starting a session

With the probe inserted, start the treatment session by increasing the intensity with the ▲ button.

The treatment countdown timer will begin.

2.5. During a session

Intensity selection

Select the stimulation intensity by using the ▲▼ buttons.

Perfect PFE has 99 intensity levels. Each press of the ▲ and ▼ buttons will adjust the intensity by 1mA. The intensity can be increased or decreased quickly by pressing and holding the ▲▼ buttons.

Prog.	Recommended intensity setting
Stress	Increase the intensity until the muscles around the vagina contract.
Urge	Increase the intensity such that it is strong but comfortable. No muscle contractions are needed.
Mixed	First 10 minutes: See Stress guidance. Second 10 minutes: See Urge guidance.
Tone	Increase the intensity until the muscles around the vagina contract.
Pain	Increase the intensity such that it is strong but comfortable. No muscle contractions are needed.

Intensity may need to be increased midway through treatment as your body gets used to the sensation.

If the sensation becomes uncomfortable, reduce the intensity.

Note: At intensity levels about 20, if the Perfect PFE detects a disconnection it will automatically

return the intensity to zero, the display will flash "LEADS" and the device will beep three times.

To increase intensity, ensure the probe is properly inserted and the lead wires are securely connected to both the probe and the device.

Warning: When the yellow LEDs in the channel output sockets light, this indicates the stimulation intensity is high enough to be harmful if not used correctly. Please use with caution.

Note: If you experience muscle cramping, pause the treatment until the symptoms subside. Then, resume the session at a lower intensity.

Stimulation output

For the Stress, Tone and first half of the Mixed programmes, the stimulation is intermittent.

The programmes have 'work' periods in which the stimulation is active and 'rest' periods with no stimulation.

This pattern ensures effective muscle engagement and adequate recovery time between contractions.

During the 'rest' periods, the intensity setting on the display will flash 00.0. The rest period will last between 5–10 seconds depending on the selected programme, before returning to the 'work period'.

During the 'work' period, the display will show the selected intensity setting, and you should feel the stimulation.

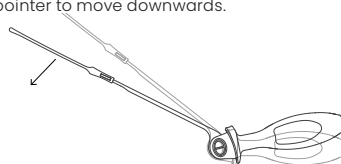
For the best results, try to voluntarily contract the pelvic floor muscles during the work periods and sustain the contractions into the rest periods.

The Urge programme and second half of the mixed programme have a constant output and do not require you to voluntarily try and contract your muscles.


Using the pointer

The probe pointer will give an indication of the strength and duration of each contraction, and whether or not the contraction is being performed correctly.

Correct muscle contractions will cause the pointer to move downwards.

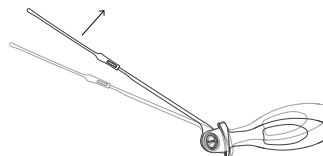Incorrect muscle contractions will cause the pointer to move upwards.

Little or no movements indicates weak muscle contractions.

Keypad lock

If you do not press any keys for 30 seconds, the keypad will lock. This is to avoid accidental changes in setting. To unlock, press the ▼ button.

Low battery

When the battery is low, the battery icons will display on the screen.

The device will automatically turn off after 2 minutes.

Replace with new AA batteries.

2.6. After a treatment

Disconnecting the device

Once the treatment timer reaches zero, the treatment session will end, and the device turns off automatically.

1. Remove the probe by holding the flange and gently pulling outwards.

Warning: Make sure the device is off before removing the probe.

Note: Do not pull on the lead wire to remove the probe, as this can cause damage.

2. Disconnect the leads from the probe and the device.
3. Place the leads and the device in the storage pouch.

Cleaning the probe

It is important the probe is cleaned after every use.

1. Clean the probe with either an alcohol-free antibacterial wipe or by washing with warm soapy water.
2. Rinse in clean water and dry thoroughly.
3. Place the probe in the storage pouch between uses.

Frequency of use

Prog.	Frequency of use	Time to improvement
Stress	Once per day	Improvements may be noticeable after 4 weeks
Urge	Once per day	Some improvements may be noticeable after 2 weeks
Mixed	Once per day	Improvements may be noticeable after 4 weeks
Tone	Twice per week	N/A – programme should be used to maintain pelvic health
Pain	Whilst experiencing pain	Relief should build up over 40 minutes and last for a few hours after use.

i Note: Time to improvements will vary from person to person.

Usage memory

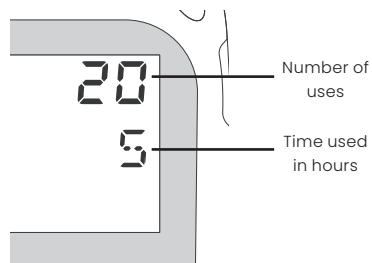
The Perfect PFE has a memory function.

To view the usage memory, press T and ▼ buttons for 3-5 seconds.

The display will show the number of times used and duration in hours used.

Press the same buttons again to return to normal controls.

To reset memory to zero, hold down the T and off buttons together for 3 to 5 seconds.

2.7. Treatment guidance summary

	Aim	Treatment time	Intensity	Frequency of use
Stress	Treat stress incontinence	20 minutes	Increase until muscles contract	Once per day
Urge	Treat urge incontinence	Default continuous. Start with 20 minutes and increase the duration if you do not notice improvements.	Increase until strong but comfortable. No muscle contractions are needed.	Once per day

Mixed Treat mixed incontinence 20 minutes

First 10 minutes: See Stress guidance
Second 10 minutes: See Urge guidance

Tone	After-care for maintaining pelvic floor health	20 minutes	Increase until muscles contract.	Twice per week
-------------	--	------------	----------------------------------	----------------

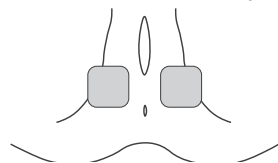
Pain Relief from chronic pelvic pain 40 minutes

Increase until strong but comfortable. No muscle contractions are needed.

2.8. Alternative electrode treatment


Alternatively, stress and urge incontinence can be treated using skin electrodes, though this method may be a less effective treatment than using a vaginal probe.

For stress incontinence, place the electrode on the skin between the anus and genitals.

Ensure the stimulation is strong enough to cause a slight contraction of the anus.

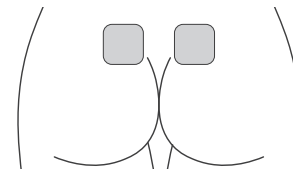For urge incontinence, place the electrode either side of the spine, close to the buttocks.

The stimulation should be strong yet comfortable.

Follow the treatment time guidance provided for vaginal probe treatment.

2.9. Alternative anal probe treatment

The TensCare X-PR13 anal probe is available for purchase and can be used to treat faecal incontinence in both men and women.

Similar to how a vaginal probe is used, this device helps manage faecal incontinence by stimulating the external anal sphincter and/or the puborectalis muscle.

To achieve optimal results, ensure that the circular electrodes are positioned just inside the sphincter.

For treatment, use the STRES or TONE programs, and adjust the intensity to the highest level that remains comfortable and pain-free. Whenever possible, try to contract the pelvic floor muscles simultaneously to enhance the therapy's effectiveness.

2.10. Kegel exercises

In addition to electrode therapy, performing Kegel exercises can significantly improve bladder control. Kegel exercises involve repeatedly contracting and relaxing the pelvic floor muscles, which can strengthen these muscles and help manage both stress and urge incontinence. Regular practice of Kegel exercises is recommended for optimal results.

- 1. Identify the right muscles:** To locate your pelvic floor muscles, try stopping urination midstream. The muscles you use for this are your pelvic floor muscles.
- 2. Get comfortable:** You can do Kegel exercises in various positions—lying down, sitting, or standing. Choose a position that feels comfortable for you.
- 3. Contract the muscles:** Tighten your pelvic floor muscles and hold the contraction for about 5 seconds.
- 4. Relax:** Release the contraction and relax the muscles for 5 seconds. Make sure to breathe normally throughout the exercise.

- 5. Repeat:** Aim for 10–15 repetitions per session. You can gradually increase the hold time and the number of repetitions as you get stronger.

Tips

- Avoid using your abdominal, leg, or buttock muscles while doing Kegels.
- Do not perform Kegels while urinating, as this can lead to incomplete emptying of the bladder over time.
- Be patient; it may take a few weeks to notice improvements.
- Perform Kegel exercises three times a day for the best results.

3. Troubleshooting

If you are experiencing issues with the device, refer to the troubleshooting guide below.

Problem	Possible causes	Solution
Display will not turn on	Flat batteries.	Replace batteries.
	Batteries inserted incorrectly.	Remove plastic wrap. Check the positive and negative ends are positioned correctly.
	Damaged springs in battery compartment.	Contact supplier.
Low battery display	Low batteries.	Replace batteries.
Controls will not work	The keypad is locked.	If the key icon is shown, press the ▼ button to unlock the device.
No stimulation sensation / LEADS showing on the display	There is a connection error. This should be noticeable by LEADS displaying on the screen. This may be due to a number of reasons:	
	Lead wire is not connected to the probe and device correctly.	Ensure the probe and device are connected correctly. To assess if the device is faulty, dampen your hand with water and a bit of salt. Firmly grip the probe, ensuring your skin covers the metal parts. Gradually increase the strength until you feel something, typically around 25.0 mA. If the LEADS alarm activates and the unit won't exceed 20.0 mA, the lead wires need replacing.
	There is a poor connection between your body and the vaginal probe.	Use a water-based lubricant such as TensCare Go Gel to improve conductivity. Cross your legs and squeeze. This can improve the probe's connection. If this helps, the contact should improve after a few weeks of use. If not, the unit may be unsuitable for you. Consult your healthcare professional for alternative options.

No stimulation sensation	Intensity level is not high enough.	Ensure you increase the intensity sufficiently. With the probe inserted, you may need to raise it to 40.0–60.0 mA. Max power is 99 mA. Adjust the intensity until you feel it.
	Reduced sensitivity in the area being treated.	You may have reduced sensitivity due to previously damaged or desensitised pudendal nerves (this can happen in childbirth or some surgical procedures). Please consult your healthcare professional.
No sensation of one side of the probe	The probe is not positioned correctly.	It's impossible for one side of the probe to malfunction while the other operates correctly. Instead, you might experience a stronger sensation on one side. To address this, try adjusting the probe's position slightly, making sure the metal plates face outward toward the hips.
Sudden change in sensation	If you stop the stimulation and continue a few minutes later, the signal will feel quite a lot stronger.	Always return the intensity to zero after disconnecting the leads or electrode pads.

If you are unable to resolve the issue, please contact our customer service team:

Email: customer.service@tenscare.co.uk

Phone: +44 1372 723434

Note: The Perfect PFE has no essential performance characteristics. Device failure will not cause unacceptable clinical risk for the user.

4. Device Care

4.1. Cleaning

It is important that the probe is cleaned before using for the first time and after each use.

1. Clean with either an alcohol-free antibacterial wipe such as TensCare wipes or by washing with warm soapy water.
2. Rinse in clean water and dry thoroughly.
3. Return the unit to the storage pouch.

Warning: Do not immerse the probe in a liquid.

Clean the device and lead wires at least once a week by wiping with a damp cloth and solution of mild soap and water. Wipe dry.

- Do not immerse the device in water.
- Do not use any other cleaning solution than soap and water.

4.2. Storage

Keep the device, probe and all accessories in the storage pouch between uses.

Storage life

- The device has no fixed shelf life.
- Unused batteries have a nominal shelf life of 3 years but will usually last longer than this.

4.3. Battery replacement

- When the batteries are low and the screen is displaying 'BATT', replace the batteries.
- Batteries should last at least 15 hours at 50 mA, 300 μ s, 50 Hz.
- The unit will work with rechargeable batteries, but the display may appear dim.

Battery Warnings

- Remove batteries from the device if it is unlikely to be used for a long period. Some types of batteries may leak corrosive fluid.
- Do not pierce, open, disassemble, or use in a humid and/or corrosive environment.
- Do not expose to temperatures over 60°C (140°F).
- Do not put, store or leave near sources of heat, in direct strong sunlight, in a high temperature location, in a pressurized container or in a microwave oven.

- Do not immerse in water or get wet.
- Do not short-circuit.
- Do not connect the device unless the battery cover is in place.
- If battery leakage occurs and comes in contact with the skin or eyes, wash thoroughly with lots of water and immediately seek medical attention.
- Keep batteries out of the reach of children to prevent them from swallowing them by mistake. If swallowed by child, contact doctor immediately.
- Never attempt to recharge an alkaline battery. Risk of explosion.
- Do not mix old, new or different types of batteries as this may lead to battery leakage or low battery indication.
- Disposal: Always dispose of batteries and device responsibly according to local government guidelines. Do not throw batteries onto a fire. Risk of explosion.

4.4. Probe and lead wire care

- Replace the probe every 6 months to maintain proper hygiene.
- Do not pull on the lead wires of the probe. This may cause damage.
- The probe is intended for single patient use only. Do not share the probe with anyone else. Improper treatment or cross-infection may occur.
- It is important the probe is cleaned after

each use. Ineffective cleaning may lead to irritation or infection.

- To ensure safety and effectiveness we recommend using TensCare probes and electrodes only.
- The life of the lead wires depends greatly on use. Always handle the leads with care. We recommend replacing the lead wires regularly (about every 6 months).

4.5. Replacement parts/ accessories

The following replacement parts are compatible with the Perfect PFE and may be ordered from TensCare at www.tenscare.co.uk or +44(0) 1372 723434.

X-VP	Liberty vaginal probe (28mm diameter)
X-VPL	Liberty loop vaginal probe (32mm diameter)
X-VPM	Liberty plus vaginal probe (32mm diameter)
X-AP	Liberty mini vaginal probe (24mm width)
X-VPLG	Gold liberty loop vaginal probe (32mm diameter)
K-GOI00	Go Gel water-based lubricant (100ml)
E-CM5050	50x50mm electrodes (x4)
E-CR5050	50x50mm Cool Relief in-fused electrodes (x4)

E-TRBG	50x50mm Blue Gel hypoallergenic electrodes (pack of 4)
L-CPT	Lead wire
B-AA	AA batteries (pack of 2)


Warning: Do not use this device with any probe other than those recommended above. Probes with smaller surface area may cause tissue irritation.

5. Technical Specification

Waveform	Symmetrical Bi-Phasic
Voltage	50V zero to peak across 500Ω, Constant voltage over 470-1500 Ohm, Constant current over 160-470 Ohm
Amplitude (over 500 Ohm load)	99.0 mA zero to peak +ve in 99 steps +/- 10%
Frequency range	4-50Hz
Pulse width range	200-300us
Output plug	Fully shielded: touch proof
Channels	Single channel
Power supply	2 x AA alkaline (two AA batteries)
Weight	75g (without batteries)
Dimensions	120 x 60 x 20 mm
Safety Classification	Internal power source. Designed for continuous use. No special moisture protection.
Service life	Device: >5 years, Probe: 6 months
Applied part	Vaginal probe, optional skin surface electrodes
Applied part classification	Type BF
IP rating	IP22
Environmental Specifications	
Operating conditions	Temperature range: 5 to 40°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 700 hPa to 1060 hPa
Storage conditions	Temperature range: -25 to +60°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 700 hPa to 1060 hPa

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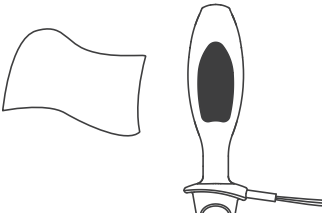Thank you for using Perfect PFE...

At TensCare, we strive to provide the best possible products and services. We value your feedback and continuously work to enhance our offerings. Understanding how our products benefit you is important to us. If you have any insights or suggestions, please get in touch.

Contact us

TensCare Ltd.
9 Blenheim Road,
Epsom, Surrey,
KT19 9BE, U.K.

+44 (0) 1372 723434

www.tenscare.co.uk

customer.service@tenscare.co.uk

Follow us

TensCare Ltd

@tenscareltd

@TensCareLtd

TensCare Limited

EU Declaration of Conformity

Tenscare Ltd herewith declare that the product meets the provisions of REGULATION (EU) 2017/745 which apply to it. The medical device has been assigned to class IIa according to Annex IV of the REGULATION (EU) 2017/745. The product concerned has been designed and manufactured under a quality management system according to Annex IX, of REGULATION (EU) 2017/745. This EU declaration of conformity is issued under the sole responsibility of the manufacturer. No "Common Specification" is applicable.

www.tenscare.co.uk

TensCare Ltd.

9 Blenheim Road, Epsom,
Surrey, KT19 9BE, U.K.

EU DoC

Complies with Medical Device
Regulation (EU) 2017/745.

Advena Ltd.

Tower Business Centre,
2nd Flr, Tower Street, Swatar,
BKR 4013, Malta

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